INSTRUCTIONS FOR USE
IFU-01/2015.version1

Open the packaging

1. Wash hands thoroughly using soap and water or use a wet wipe or cleansing gel.
2. Always check the manufacturer's instructions and follow the guidance given by your healthcare professional.
3. If you have a self-lubricating catheter, soak in water as instructed.
4. If you have a non-lubricating catheter, you might need to use a gel to coat the catheter.
5. Adopt the position that you find most comfortable in order to catheterise.
6. Visually inspect the catheter for any damage and use it immediately.
7. Slowly and carefully insert the catheter into the urethra or your reconstruction channel. You will know you are in the bladder as urine will drain from the catheter.
8. Once the bladder is empty slowly remove the catheter.
9. After use put the catheter back into the packaging before disposing of responsibly.
10. Wash hands again with soap and water or use a wet wipe or cleansing gel.

How often should I use the catheter?

This depends on how much fluid you drink but most people need to empty their bladder every 4-6 hours when they are awake.

Potential side effects

A small amount of blood in the urine may occur, especially in the beginning due to irritation of the mucous membrane of the urethra.

Contact your doctor

Contact your doctor if you experience symptoms of a urinary tract infection: **Fever, Discomfort or pain when urinating, Frequent need to urinate, Blood in the urine.**

Manufactured using DEHP FREE PVC. Contraindications: Do not use if sensitive to PVC. Use the catheter only as prescribed by a doctor and trained by a Health professional. Precautions: Sterility can only be guaranteed for packaging which is intact and sealed. Storage: Keep dry and away from direct sunlight. Indication: Transient bladder catheterisation by introduction through urethra or stoma.

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